

Product Information Guide

Spring 2026



Table of Contents

Devices	4-14
AirSense™ 11 PAP devices	5
AirCurve™ 11 bilevel devices	6
Air11™ accessories	8
AirSense™ 10 PAP devices	9
AirCurve™ 10 bilevel devices	10
Air10™ accessories	12
AirMini™	13
Astral™ series ventilators	14
Masks	15-33
Resmed PAP masks	16
AirFit™ F40	19
AirFit™ N30	20
AirFit™ P10	21
AirTouch™ F30i Comfort	22
AirTouch™ F30i Clear	23
AirFit™ F30i	24
AirFit™ X30i	25
AirTouch™ N30i	26
AirFit™ N30i	27
AirFit™ P30i	28
AirFit™ F20	29
AirTouch™ F20	30
AirFit™ N20	31
AirTouch™ N20	32
Pixi™	33
Cleaning Resmed masks	35
Limited warranty information	36
Resmed Online Store	37
Resmed myAir™	39



Devices



AirSense™ 11 PAP devices

Resmed AirSense™ 11 delivers enhanced PAP therapy with a sleek design, intuitive touchscreen, and integrated humidifier for maximum comfort. Paired with the myAir™ app, it provides personalized insights and guidance designed to support therapy adherence and outcomes.



	AirSense 11 AutoSet™ Enhanced self-adjusting AutoSet therapy device	AirSense 11 CPAP CPAP therapy device
Operating modes and pressures		
CPAP	✓	✓
AutoSet or AutoSet for Her	✓	
Operating pressure range	4–20 cm H ₂ O	4–20 cm H ₂ O
AutoSet mode default pressure	4–20 cm H ₂ O	
Features		
AutoSet Response comfort setting	✓	
CSA, RERA and CSR detection	✓	
AutoRamp with sleep onset detection	✓	✓
Easy-Breathe waveform with EPR	✓	✓
SmartStart™/Stop feature	Individually toggleable	Individually toggleable
Oxygen use	Up to 15 L/min	Up to 15 L/min
Remote therapy management in Airview™	✓	✓
Patient access to Resmed myAir™†	✓	✓
Bluetooth® connectivity	✓	✓
Over-the-air updates	✓	✓
Humidification		
Standard heated humidifier	Integrated	Integrated
ClimateLineAir™ 11 heated tubing	Optional	Optional
ClimateLineAir™ 11 Oxy heated tubing	Optional	Optional
Data transfer on SD card		
Detailed data	30 sessions	
High-resolution flow	30 sessions	
Compliance and summary data	365 sessions	365 sessions

Model	Part number (US)	Part number (Canada)	HCPC	Replacement schedule**
AirSense 11 AutoSet • With HumidAir™ 11 • With HumidAir 11 and ClimateLineAir 11	39000 39001	39006 39007	E0601	Every 5 years
AirSense 11 Elite • With HumidAir 11 • With HumidAir 11 and ClimateLineAir 11	39004 39005	39010 39011	E0601	Every 5 years
AirSense 11 CPAP • With HumidAir 11 • With HumidAir 11 and ClimateLineAir 11	39002 39003	39008 39009	E0601	Every 5 years

† The myAir by Resmed app is available in English and Spanish in the US and in English and French in Canada. Some features of myAir are only available in the myAir app and with Air 11 devices.

** The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirCurve™ 11 bilevel devices

Resmed AirCurve™ 11 devices offer enhanced bilevel therapy tailored to different needs: VAuto provides auto-adjusting Inspiratory Positive Airway Pressure (IPAP) and Expiratory Positive Airway Pressure (EPAP) support for obstructive sleep apnea (OSA), while Adaptive-Servo Ventilation (ASV) responds to complex breathing patterns for central and mixed apneas. Both feature heated humidification, integrated connectivity, and intelligent algorithms for personalized comfort throughout the night.



	AirCurve11 VAuto Treats patients with obstructive sleep apnea and may help those struggling to adhere to PAP therapy	AirCurve 11 ASV* Treats patients presenting with central sleep apnea, obstructive sleep apnea, mixed apneas or periodic breathing
Operating modes and pressures		
CPAP	✓	✓
VAuto	✓	
Spontaneous (S)	✓	
ASV*		✓
ASVAuto		✓
Operating pressure range ‡	4–25 cm H ₂ O	4–25 cm H ₂ O
Features		
CSA detection	✓	
Ramp	✓	✓
Easy-Breathe waveform with EPR	✓	
Vsync automatic leak management	✓	✓
Auto-adjusts pressure§ for OSA events	✓	When in ASVAuto
Optional integrated oximetry	✓	✓
3 m tubing	✓	✓
Adaptive minute ventilation targeting (90% of recent 3-minute average; 15 bpm initial back-up rate; starting target 4 L/min, weighted over last 4–6 breaths)		✓
Remote therapy management in Airview™	✓	✓
Integrated cellular module	✓	✓
Patient access to Resmed myAir™†	✓	✓
Bluetooth® connectivity	✓	✓
Over-the-air updates	✓	✓
SmartStart™/Stop feature	Individually toggleable	Individually toggleable
Oxygen use	Up to 15 L/min	Up to 15 L/min
Humidification		
Standard heated humidifier	Integrated	Integrated
ClimateLineAir™ 11 heated tubing	Optional	Optional
ClimateLineAir™ 11 Oxy heated tubing	Optional	Optional
Data transfer on SD card		
Detailed data	30 sessions	30 sessions
High-resolution flow	30 sessions	30 sessions
Compliance and summary data	365 sessions	365 sessions

Model	Part number (US)	Part number (Canada)	HCPC	Recommended replacement schedule**
AirCurve 11 VAuto with HumidAir™ 11	39018		E0470	Every 5 years
AirCurve 11 VAuto with HumidAir 11 and ClimateLineAir 11	39019	39020	E0470	Every 5 years
AirCurve 11 ASV* with HumidAir 11	39012		E0471	Every 5 years
AirCurve 11 ASV* with HumidAir 11 and ClimateLineAir 11	39013	39023	E0471	Every 5 years

* ASV therapy is contraindicated in patients with chronic, symptomatic heart failure (NYHA 2–4) with reduced left ventricular ejection fraction (LVEF ≤ 45%) and moderate to severe predominant central sleep apnea.
† The myAir by Resmed app is available in English and Spanish in the US and in English and French in Canada. Some features of myAir are only available in the myAir app and with Air11 devices.
‡ Operating pressures may vary depending on the mode. Please refer to the user guide for the technical specifications.
§ This feature is dependent on the mode.
** The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.



Air11™ accessories

	Accessory	Part number	HCPC	Recommended replacement schedule**
Humidification				
	HumidAir™ 11 standard tub	39100	A7046	Every 6 months
	HumidAir 11 cleanable tub	39101	A7046	Every 6 months
	ClimateLineAir™ 11 heated tubing	39102	A4604	Every 3 months
	ClimateLineAir™ 11 Oxy heated tubing	39232	A4604	Every 3 months
	SlimLine™ tubing	36810	A7037	Every 3 months
	6' standard tubing	14994	A7037	Every 3 months
	6'6" standard tubing	14987		
	6'6" autoclavable standard tubing	14986		
	3m tubing	14999		
	Air11 side cover	39226		
	Air11 air outlet	39220		
Filters				
	Air11 standard filters, 1 pk	39300	A7038	2 per month
	Air11 standard filters, 2 pk	39301		
	Air11 standard filters, 12 pk	39302		
	Air11 standard filters, 50 pk	39303		
	Air11 hypoallergenic filters, 1 pk	39304		
	Air11 hypoallergenic filters, 2 pk	39305		
	Air11 hypoallergenic filters, 12 pk	39306		
	Air11 hypoallergenic filters, 50 pk	39307		
	Filter door replacement	29108		
Power, data management and oximetry				
	AirSense™ 11 power supply unit, 65W	39205		
	AirCurve™ 11 power supply unit, 90W	39206		
	Air11 DC/DC converter	39231		
	Air11 SD card with envelope, 1 pk	39228		
	AirSense 11 SD card with envelope, 10 pk	39229		
	Air11 SD card door	39225		
	AirCurve 11 SD card door	39233		
	Air11 BLE WristOx-2	7079666		
	Air11 expandable wrist band	7079667		
	Air11 small soft sensor, WO2 connector	7079814		
	Air11 medium soft sensor, WO2 connector	7079815		
	Air11 single-use wrist band, 15 pk	7079668		
	Air11 large soft sensor, WO2 connector	7079816		
	Air11 single-use sensor and wrist band, 15 pk	70710011		
Faceplates, travel bags and quick start guides				
	Air 11 faceplate	29106		
	Air11 travel bag	39221		
	AirSense 11 user guide	398133		
	AirCurve 11 user guide	398195		
	Air11 quick start guide	398122		

** The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirSense™ 10 PAP devices

Resmed AirSense™ 10 devices provide everyday PAP therapy at home or in hospital, featuring integrated humidification, built-in wireless connectivity, and AutoRamp™ with sleep-onset detection, designed to help deliver reliable, easy-to-use treatment for obstructive sleep apnea.



	AirSense 10 AutoSet™ Enhanced self-adjusting AutoSet therapy device	AirSense 10 CPAP CPAP therapy device
Operating modes and pressures		
CPAP	✓	✓
AutoSet	✓	
Operating pressure range	4–20 cm H ₂ O	4–20 cm H ₂ O
AutoSet mode default pressure	4–20 cm H ₂ O	
Features		
AutoSet Response comfort setting	✓	
CSA, RERA and CSR detection	✓	
AutoRamp with sleep onset detection	✓	✓
Easy-Breathe waveform with EPR	✓	✓
SmartStart™/Stop feature	✓	✓
Remote therapy management in Airview™	✓	✓
Integrated cellular module	✓	✓
Patient access to Resmed myAir™†	✓	✓
Oxygen use	Up to 4 L/min	Up to 4 L/min
Humidification		
Standard heated humidifier	Integrated	Integrated
ClimateLineAir™ 10 heated tubing	Optional	Optional
ClimateLineAir™ Oxy heated tubing	Optional	Optional
Data transfer on SD card		
Detailed data	30 sessions	365 sessions
High-resolution flow	30 sessions	
Compliance and summary data	365 sessions	

Model	Part number (US)	Part number (Canada)	HCPC	Recommended replacement schedule**
AirSense 10 AutoSet • With HumidAir™ • With HumidAir and ClimateLineAir	37207	37402	E0601	Every 5 years
	37208	37403		
AirSense 10 Elite • With HumidAir • With HumidAir and ClimateLineAir	37205	37400	E0601	Every 5 years
	37206	37401		
AirSense 10 CPAP • With HumidAir • With HumidAir and ClimateLineAir	37203	37398	E0601	Every 5 years
	37204	37399		

† The myAir by Resmed app is available in English and Spanish in the US, and in English and French in Canada. Some features of myAir are only available in the myAir app and with Air11 devices.

** The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirCurve™ 10 bilevel PAP devices

Resmed AirCurve™ 10 devices provide dual-level support — inspiratory positive airway pressure (IPAP) and expiratory positive airway pressure (EPAP) — with integrated humidification and wireless connectivity, designed to help deliver comfortable, adaptive therapy for obstructive sleep apnea and ventilation needs at home or in hospital use.



	AirCurve 10 VAuto Treats patients with obstructive sleep apnea and may help those struggling to adhere to PAP therapy	AirCurve 10 ASV* Treats patients presenting with central sleep apnea, obstructive sleep apnea, mixed apneas and periodic breathing	AirCurve 10 ST Treats patients with obstructive sleep apnea	AirCurve 10 ST-A Treats patients with ventilatory insufficiency or obstructive sleep apnea
Operating modes and pressures				
CPAP	✓	✓	✓	✓
VAuto	✓			
Spontaneous (S)	✓		✓	✓
Spontaneous/Timed (ST) and Timed (T)			✓	✓
PAC and iVAPS				✓
ASV*		✓		
ASVAuto*		✓		
Operating pressure range†	3–25 cm H ₂ O	4–25 cm H ₂ O	3–25 cm H ₂ O	3–30 cm H ₂ O
Features				
Optional integrated oximetry	✓	✓	✓	✓
Remote therapy management in Airview™	✓	✓	✓	✓
Integrated cellular module	✓	✓	✓	✓
Vsync automatic leak management	✓	✓	✓	✓
Leak alert	✓	✓	✓	
3 m tubing	✓	✓	✓	✓
Ramp	✓	✓	✓	✓
Auto-adjusts pressure‡ for OSA events	✓	When in ASVAuto		
Patient access to Resmed myAir™†	✓	✓	✓	✓
SmartStart™/Stop feature	✓	✓	✓	✓
Adjustable trigger and cycle sensitivity	✓		✓	✓
TiControl (Ti Max/Ti/Min)	✓		✓	✓
CSA detection	✓			
Easy-Breathe waveform with EPR	✓			
Adaptive minute ventilation targeting (90% of recent 3-minute average; 15 bpm initial back-up rate; starting target 4 L/min, weighted over last 4–6 breaths)		✓		
Device alarms		✓		✓
Oxygen use	4 L/min	15 L/min (CPAP, ASV*), 4 L/min (ASVAuto)	Up to 15 L/min	Up to 15L/min, 4 L/min (iVAPS)

	AirCurve 10 VAuto Treats patients with obstructive sleep apnea and may help those struggling to adhere to PAP therapy	AirCurve 10 ASV* Treats patients presenting with central sleep apnea, obstructive sleep apnea, mixed apneas and periodic breathing	AirCurve 10 ST Treats patients with obstructive sleep apnea	AirCurve 10 ST-A Treats patients with ventilatory insufficiency or obstructive sleep apnea
Humidification				
Standard heated humidifier	Integrated	Integrated	Integrated	Integrated
ClimateLineAir™ heated tubing	Optional	Optional	Optional	Optional
ClimateLineAir™ Oxy heated tubing	Optional	Optional	Optional	Optional
Data transfer on SD card				
Detailed data	30 sessions	30 sessions	30 sessions	30 sessions
High-resolution flow	30 sessions	30 sessions	30 sessions	30 sessions
Compliance and summary data	365 sessions	365 sessions	365 sessions	365 sessions

Model	Part number (US)	Part number (Canada)	HCPC	Recommended replacement schedule**
AirCurve 10 VAuto With HumidAir™	37211	37406	E0470	Every 5 years
AirCurve 10 VAuto with HumidAir and ClimateLineAir	37212		E0470	Every 5 years
AirCurve 10 ASV* with HumidAir	37215	37410	E0471	Every 5 years
AirCurve 10 ASV* with HumidAir and ClimateLineAir	37216		E0471	Every 5 years
AirCurve 10 ST with HumidAir	37306	37412	E0471	Every 5 years
AirCurve 10 ST with HumidAir and ClimateLineAir	37307		E0471	Every 5 years
AirCurve 10 ST-A with HumidAir	28210	28224	E0471	Every 5 years
AirCurve 10 ST-A with HumidAir and ClimateLineAir	28211		E0471	Every 5 years

* ASV therapy is contraindicated in patients with chronic, symptomatic heart failure (NYHA 2–4) with reduced left ventricular ejection fraction (LVEF ≤ 45%) and moderate to severe predominant central sleep apnea.

† The myAir by Resmed app is available in English and Spanish in the US, and in English and French in Canada. Some features of myAir are only available in the myAir app and with AirIt devices.

‡ Operating pressures may vary depending on the mode. Please refer to the user guide for the technical specifications.

§ This feature is dependent on the mode.

** The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

Air10™ accessories

	Accessory	Part number	HCPC	Recommended replacement schedule**
Humidification				
	HumidAir™ standard tub	37299	A7046	Every 6 months
	HumidAir cleanable tub	37479	A7046	Every 6 months
	ClimateLineAir™ heated tubing	37296	A4604	Every 3 months
	ClimateLineAir™ Oxy heated	37357	A4604	Every 3 months
	SlimLine™ tubing	36810	A7037	Every 3 months
	6' standard tubing	14994	A7037	Every 3 months
	6'6" standard tubing	14987		
	6'6" autoclavable standard tubing	14986		
	3m tubing	14999		
	Air10 Side Cover, Gray	37335		
	Air10 Side Cover, Charcoal	37303		
Filters				
	Air10™ standard filters, 1 pk	36850	A7038	Every 2 weeks
	Air10 standard filters, 2 pk	36851		
	Air10 standard filters, 12 pk	36852		
	Air10 standard filters, 50 pk	36853		
	Air10 hypoallergenic filters, 1 pk	36855		
	Air10 hypoallergenic filters, 2 pk	36856		
	Air10 hypoallergenic filters, 12 pk	36857		
	Air10 hypoallergenic filters, 50 pk	36858		
	Air10 Filter Cover, Gray	19768		
Air10 Filter Cover, Charcoal	19729			
Power, data management and oximetry				
	Air10 power supply unit	37474		
	Air10 DC/DC converter	37297		
	RPS II-V2 kit for Air10	24702		
	RPS II-V2 battery	24705		
	RPS II DC cable for Air10	37343		
	RPS II-V2 adapter for Air10	24706		
	Air10 SD card protective folder, with card, 1 pk	37378		
	Side panel SD card door, gray (AirSense 10 AutoSet for Her, AirCurve 10 ST)	19766		
	Side panel SD card door, charcoal (AirSense 10 AutoSet and Elite, AirCurve 10 ST-A)	19728		
	Air10 oximetry kit: Module, XPOD, XPOD clip and reusable soft sensor (medium)	370004		
	Air10 oximetry kit module	37302		
	XPOD LP oximeter	22374		
	XPOD LP clip	22371		
	Reusable soft sensor (medium)	707560		
Faceplates, travel bags and quick start guides				
	Air10 faceplate, silver	19736		
	Air10 faceplate, white with gray	19735		
	Air10 faceplate, black	19734		
	Air10 travel bag	37304		
	Air10 travel bag for her	37305		

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirMini™

The Resmed AirMini™ is an AutoSet™ device for PAP users on the go. As one of the smallest PAP devices on the market,* it delivers Resmed's trusted AutoSet therapy in a compact, travel-ready design. With the AirMini app allowing users to view nightly sleep scores, adjust comfort settings, and track progress from their smart device.



Device and mask kits	Part number (US)
AirMini	38113
AirFit™ N30 Mask Kit for AirMini (mask included)	38878
AirFit P10 Mask Kit for AirMini (mask included)	38824
AirFit/AirTouch™ N20 Setup Pack for AirMini (mask not included)	38823
AirFit/AirTouch F20 Setup Pack for AirMini (mask not included)	38011

	Accessory	Part number
For nasal and nasal pillows masks		
	HumidX™, 3 pk	38809
	HumidX, 6 pk	38810
	HumidX, 50 pk	38808
	HumidX Plus, 3 pk	38812
	HumidX Plus, 6 pk	38813
	HumidX Plus, 50 pk	38811
For full face masks		
	HumidX F20, 3 pk	38012
	HumidX F20, 6 pk	38013
Travel		
	AirMini travel bag	38840
	AirMini travel case	38841
Filters		
	AirMini filter, 1 pk	38814
	AirMini filter, 2 pk	38815
	AirMini filter, 12 pk	38816
	AirMini filter, 50 pk	38837
Mount system		
	AirMini mount system	38842
Power		
	AirMini DC/DC converter	38839
	AirMini 20W AC adapter	38828

*Based on available user guide data from Resmed AirMini, Philips™ DreamStation® Go, Breas Z2® Auto System, Transcend® Micro™ and Z1 Auto®. The physical dimensions of AirMini are 5.4" (W) x 3.3" (D) x 2.1" (H).

Astral™ series ventilators

Resmed Astral™ series ventilators deliver invasive and non-invasive life-support ventilation in a portable design with integrated connectivity and long battery life, helping support adult and pediatric patients across home and hospital care settings.



	Astral100	Astral150
Features		
Access to Resmed AirView™ via Resmed Connectivity Module (RCM) ¹	✓	✓
8 hours of backup internal battery power ²	✓	✓
Big Button technology	✓	✓
Easy five-step setup	✓	✓
Responsive algorithms for adaptive, personalized ventilation as disease progresses	✓	✓
FiO2 monitoring		✓
Double limb circuit capabilities		✓
Additional program setting capabilities		✓

Model	Part number (US)	Part number (Canada)	HCPC	Recommended replacement schedule**
Astral100	27001	27001	EO465 (invasive) EO466 (non-invasive)	Every 8 years
Astral150	27003	27003	EO465 (invasive) EO466 (non-invasive)	Every 8 years

Accessories	Part number (US)	Part number (Canada)	HCPC	Recommended replacement schedule**
Astral external battery	27919	27919		
Resmed connectivity module	27206	27201		
Mouthpiece ventilation circuit support arm	27955	27955		
Homecare Stand (with Astral top plate and base plate)	279001	279001		
Table stand	27915	27915		
Astral SlimFit mobility bag	27949	27949		
Astral mobility bag	27917	27917		
Astral carry bag	27945	27945		

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

¹When Astral is paired with the Resmed Connectivity Module (RCM), wireless connectivity is enabled, allowing key patient therapy data to be transmitted directly from Astral to the Resmed secure, cloud-based management system, AirView.

²An internal battery provides up to 8 hours of extra backup power if needed. A maximum of two external batteries have the capacity to deliver up to 16 hours of continuous, portable power when connected to Astral.

Masks



Resmed PAP masks

Resmed offers a range of full face, nasal and nasal pillows masks designed to deliver effective therapy, grouped into three categories.



Minimalist masks | Small. Light. Simple.



AirFit™ F40*
Full face mask



AirFit™ N30
Nasal cradle mask



AirFit™ P10
Nasal pillows mask
AirFit P10 for Her

Freedom masks | Sleep and move comfortably.



AirTouch™ F30i*
Full face mask
COMFORT



AirTouch™ F30i*
Full face mask
CLEAR



AirFit™ F30i
Full face mask
AirFit F30i Non Magnetic



AirFit™ X30i*
Oral nasal mask



AirTouch™ N30i
Nasal cradle mask



AirFit™ N30i
Nasal cradle mask



AirFit™ P30i
Nasal pillows mask

*These masks contain magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

Versatile fit masks | Designs that fit nearly every face.^{1,2,3,4}



AirFit™ F20*
Full face mask
AirFit F20 for Her¹
AirFit F20 Non Magnetic



AirTouch™ F20*
Full face mask
AirTouch F20 for Her¹



AirFit™ N20*
Nasal mask
AirFit N20 for Her²



AirTouch™ N20
Nasal mask
AirTouch N20 for Her



*These masks contain magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

¹Resmed AirFit F20 Australian fitting study of 27 existing CPAP patients, conducted between March–April 2016.

²Resmed AirFit F20 USA fitting study of 34 existing CPAP patients, conducted April 2016.

³Resmed AirFit F20 EU and APAC fitting study of 90 existing CPAP patients, conducted June 2016.

⁴Resmed AirFit N20 international fitting study of 159 existing CPAP patients, conducted Nov 2015.

AirFit™ F40

Minimalist Freedom Versatile Fit

AirFit™ F40 is an UltraCompact under-the-nose full face mask, designed to provide less coverage on the face and featuring an AdaptiSeal™ total soft silicone cushion that adapts to different face shapes for a secure, comfortable fit.

Full face mask



Components



Cushion



Frame



Quick-release elbow with short tube



Headgear



How to fit the AirFit F40 | For complete user instructions, refer to the AirFit F40 user guide.

1



Place the cushion under the nose and over the mouth, ensuring it sits comfortably against the face. With the Resmed logo on the headgear facing up, pull the headgear over the head, positioning the headgear on the top of the head.

2



Bring the lower headgear straps under the ears and attach the magnetic clips* to the frame. If needed, feed longer hair through the back of the headgear.

3



Undo the fastening tabs on the upper headgear straps and pull evenly. Repeat with the lower headgear straps.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, cushion and headgear) • Small headgear—medium cushion • Standard headgear—small wide/medium/large cushion • Large headgear—medium cushion	64603 64600/64601/64602 64604	A7030 + A7035	-
Frame system (includes frame and cushion) • Small wide • Medium • Large	64607 64608 64609	A7030	Every 3 months
Cushion • Small wide • Medium • Large	64627 64628 64629	A7031	Every month
Headgear (includes magnetic clips*) • Small • Standard • Large	64633 64634 64635	A7035	Every 6 months

*The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirFit™ N30

AirFit™ N30 features a lightweight design with a nasal cradle cushion that sits under the nose to minimize facial contact, supporting a comfortable, unobtrusive fit with a simple three-part design for easy use and cleaning.

Minimalist


Freedom

Versatile Fit

Nasal cradle mask



Components

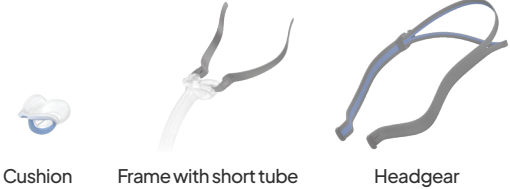

How to fit the AirFit N30 | For complete user instructions, refer to the AirFit N30 user guide.



Ensure the cushion size and the left (L) and right (R) indicators are facing forward. Place the cushion under the nose and pull the headgear over the head, positioning the top strap on the crown of the head.



Briefly move the mask away from the face and place it back to ensure a comfortable fit. If needed, feed longer hair through the back of the headgear.



Adjust the split headgear straps through the buckle for a comfortable seal. Do not overtighten.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, cushion and headgear) • Small • Medium • Small wide	64222 64223 64224	A7034 + A7035	-
Frame system (includes frame and cushion) • Small • Medium • Small wide	64210 64211 64212	A7034	Every 3 months
Cushion • Small • Medium • Small wide	64213 64214 64215	A7032	Every 2 weeks
Headgear • Standard	64216	A7035	Every 6 months

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirFit™ P10

AirFit P10 for Her

Minimalist

Freedom

Versatile Fit

Nasal pillows mask



Components



How to fit the AirFit P10 | For complete user instructions, refer to the AirFit P10 user guide.



Ensure the pillow size and the left (L) and right (R) indicators are correctly facing. Place the nasal pillows into the nostrils and pull the headgear over the head, positioning the top strap on the top of the head.



Briefly move the mask away from the face and place it back to ensure the nasal pillows fit securely and upright. If needed, feed longer hair through the back of the headgear.



Adjust the headgear straps evenly for a comfortable fit. Do not overtighten.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, three pillows sizes and headgear) • Starter pack, AirFit P10—small/medium/large pillows • Starter pack, AirFit P10 for Her—Extra small/small/medium pillows	62900 62910	A7034 + A7035	-
Frame system (includes frame and pillows) • Extra small • Small • Medium • Large	62920 62921 62922 62923	A7034	Every 3 months
Pillows • Extra small • Small • Medium • Large	62930 62931 62932 62933	A7033	Every 2 weeks
Headgear • AirFit P10 • AirFit P10 for Her	62935 62936	A7035	Every 6 months

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirTouch™ F30i Comfort

The Resmed AirTouch™ F30i Comfort features a top-of-head tube connection, fabric-wrapped frame, and ComfiSoft™ silicone-coated fabric cushion with AdaptiSeal™ technology, designed to adapt to the face and deliver softness at every touch for all-around comfort.

Components



Minimalist

Freedom

Versatile Fit



How to fit the AirTouch F30i Comfort | For complete user instructions, refer to the AirTouch F30i Comfort user guide.



Place the cushion under the nose and ensure it sits comfortably against the face. With the Resmed logo on the headgear facing up, pull the headgear and frame over the head.



Bring the lower headgear straps under the ears and attach the magnetic clips* to the frame. If needed, feed longer hair through the back of the headgear.



Undo the fastening tabs on the upper headgear straps and pull evenly. Repeat with the lower headgear straps.

Ordering information

Product	Part number	Recommended replacement schedule**
Complete mask system (includes frame, cushion and headgear)		
• Small frame—medium cushion	62442	—
• Standard frame—small wide/medium/large cushion	62443/62444/62445	
• Large frame—medium cushion	62446	
Frame system (includes frame and cushion)		
• Small frame—small wide/medium cushion	62457/62458	Every 3 months
• Standard frame—small wide/medium/large cushion	62460/62461/62462	
• Large frame—medium/large cushion	62464/62465	
Cushion		
• Small wide	62489	Every month
• Medium	62490	
• Large	62491	
Headgear (includes magnetic clips*)		
• Standard	63372	Every 6 months

*This mask contains magnets that may interfere with certain implants of medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirTouch™ F30i Clear

The Resmed AirTouch™ F30i Clear* features a top-of-head tube connection and flexible frame with a ComfiSoft™ silicone-coated fabric cushion using AdaptiSeal™ technology, designed to adapt to the face and provide softness where patients feel it most — at the seal.

Minimalist

Freedom

Versatile Fit

Components



How to fit the AirTouch F30i Clear | For complete user instructions, refer to the AirTouch F30i Clear user guide.



Place the cushion under the nose and ensure it sits comfortably against the face. With the Resmed logo on the headgear facing up, pull the headgear and frame over the head.



Bring the lower headgear straps under the ears and attach the magnetic clips** to the frame. If needed, feed longer hair through the back of the headgear.



Undo the fastening tabs on the upper headgear straps and pull evenly. Repeat with the lower headgear straps.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule†
Complete mask system (includes frame, cushion and headgear)			
• Small frame—medium cushion	62404	A7030 + A7035	—
• Standard frame—small wide/medium/large cushion	62405/62406/62407		
• Large frame—medium cushion	62408		
Frame system (includes frame and cushion)			
• Small frame—small wide/medium cushion	62477/62478	A7030	Every 3 months
• Standard frame—small wide/medium/large cushion	62480/62481/62482		
• Large frame—medium/large cushion	62484/62485		
Cushion			
• Small wide	62489	A7031	Every month
• Medium	62490		
• Large	62491		
Headgear (includes magnetic clips*)			
• Standard	63372	A7035	Every 6 months

*AirTouch F30i Clear is available for sale only in the US.

**This mask contains magnets that may interfere with certain implants of medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

†The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirFit™ F30i

Minimalist

Freedom

Versatile Fit

AirFit F30i Non Magnetic

The Resmed AirFit™ F30i features a top-of-head tube connection, flexible frame, and UltraCompact under-the-nose cushion designed to reduce nasal-bridge irritation and helps deliver dependable full face therapy with less bulk.

Full face mask



Components



Cushion



Frame with quick-release elbow



Headgear



How to fit the AirFit F30i | For complete user instructions, refer to the AirFit F30i user guide.



Place the cushion under the nose and ensure it sits comfortably against the face. With the Resmed logo on the headgear facing up, pull the headgear and frame over the head.



Bring the lower headgear straps under the ears and attach the mask clips* to the frame. If needed, feed longer hair through the back of the headgear.



Undo the fastening tabs on the upper headgear straps and pull evenly. Repeat with the lower headgear straps.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, cushion and headgear) - Small frame—small cushion - Standard frame—small/medium/wide cushion - Large frame—medium cushion Non Magnetic, small frame—small cushion Non Magnetic, standard frame—small/medium/wide cushion Non Magnetic, large frame—medium cushion	63330 63331/63332/63333 63334 63303 63304/63309/63315 63316	A7030 + A7035	—
Frame system (includes frame and cushion) - Small frame—small/medium/small wide/wide cushion - Standard frame—small/medium/small wide/wide cushion - Large frame—small/medium/small wide/wide cushion	63355/63356/63357/63358 63359/63360/63361/63362 63363/63364/63365/63366	A7030	Every 3 months
Cushion - Small - Small wide - Medium - Wide	63350 63352 63351 63353	A7031	Every month
Headgear (includes magnetic clips*) - Standard Non Magnetic—standard	63372 63317	A7035	Every 6 months

*AirFit F30i contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings. AirFit F30i Non Magnetic is a non-magnetic variant available to order.

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirFit™ X30i

Minimalist

Freedom

Versatile Fit

The Resmed AirFit™ X30i features a top-of-head tube connection and flexible frame, combining a nasal pillows cushion for stability with an oral cushion for coverage to help deliver a dependable full face seal during movement throughout the night.

Oral nasal mask



Components



Cushion



Pillows



Frame with quick-release elbow



Headgear



How to fit the AirFit™ X30i | For complete user instructions, refer to the AirFit X30i user guide.



Place the oral cushion over the mouth and position the nasal pillows into the nostrils. With the Resmed logo on the headgear facing up, pull the headgear and frame over the head.



Bring the lower headgear straps under the ears and attach the magnetic clips* to the frame. If needed, feed longer hair through the back of the headgear.



Undo the fastening tabs on the upper headgear straps and pull evenly. Repeat with the lower headgear straps.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, oral cushion, three pillows sizes and headgear) - Starter pack, small frame—small/medium/large pillows with oral cushion - Starter pack, standard frame—small/medium/large pillows with oral cushion - Starter pack, large frame—small/medium/large pillows with oral cushion	64300 64301 64350	A7027 + A7035	—
Frame system (includes frame, oral cushion, pillows and headgear) - Small frame—small/medium/large pillows with oral cushion - Standard frame—small/medium/large pillows with oral cushion - Large frame—small/medium/large pillows with oral cushion	64302/64303/64304 64305/64306/64307 64351/64352/64353	A7030	Every 3 months
Cushions - Small pillows - Medium pillows - Large pillows - Oral cushion	64309 64310 64311 64308	pillows cushion: A7029 oral cushion: A7028	Every 2 weeks for each cushion type*
Headgear (includes magnetic clips*) Standard	63372	A7035	Every 6 months

*This mask contains magnets that may interfere with certain implants of medical devices. Please refer to the User Guide for complete labeling information, including magnet contraindications and warnings.

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirTouch™ N30i

The Resmed AirTouch™ N30i features a top-of-head tube connection, ComfiSoft™ silicone-coated fabric cushion, and a fabric-wrapped SpringFit™ frame designed to adjust to the face and deliver breathable, moisture-wicking¹ softness for all-night comfort.

Minimalist

Freedom

Versatile Fit

Nasal cradle mask



Components



How to fit the AirTouch N30i | For complete user instructions, refer to the AirTouch N30i user guide.



Pull the frame and headgear over the head until the elbow is positioned on top and the headgear is behind the head. Place the cushion under the nose.



Tighten or loosen the headgear until the cushion fits comfortably under the nose. If needed, feed longer hair through the back of the headgear.



Squeeze the side buttons on the elbow and detach from the frame. Connect the air tubing from the device to the elbow, then reattach to the frame.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, cushion and headgear) - Starter pack, small frame—small wide/medium cushion - Starter pack, standard frame—small wide/medium cushion - Small frame—small wide/medium/large cushion - Standard frame—small wide/medium/large cushion	62301 62300 62313/62314/62315 62310/62311/62312	A7034 + A7035	—
Frame system (includes frame and cushion) - Small frame—small wide/medium/large cushion - Standard frame—small wide/medium/large cushion	62319/62320/62321 62316/62317/62318	A7034	Every 3 months
Cushion - Small wide - Medium - Large	62330 62331 62332	A7032	Every 2 weeks
Headgear Standard	63814	A7035	Every 6 months

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

¹Resmed internal testing for the moisture-wicking properties of fabric, ID: A5498268

AirFit™ N30i

The Resmed AirFit™ N30i features a top-of-head tube connection and SpringFit™ frame that adapts to the face, with a nasal cradle cushion that sits under the nose to help deliver a secure, comfortable seal in any sleep position.

Minimalist

Freedom

Versatile Fit

Nasal cradle mask



Components



How to fit the AirFit N30i | For complete user instructions, refer to the AirFit N30i user guide.



Pull the frame and headgear over the head until the elbow is positioned on top and the headgear is behind the head. Place the cushion under the nose.



Tighten or loosen the headgear until the cushion fits comfortably under the nose. If needed, feed longer hair through the back of the headgear.



Squeeze the side buttons on the elbow and detach from the frame. Connect the air tubing from the device to the elbow, then reattach to the frame.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, three cushion sizes and headgear) - Starter pack, small frame—small/medium/small wide cushion - Starter pack, standard frame—small/medium/small wide cushion	63801 63800	A7034 + A7035	—
Frame system (includes frame and cushion) - Small frame—small/medium/wide/small wide cushion - Standard frame—small/medium/wide/small wide cushion	63803/63805/63807/63809 63802/63804/63806/63808	A7034	Every 3 months
Cushion - Small - Small wide - Medium - Wide	63813 63810 63811 63812	A7032	Every 2 weeks
Headgear Standard	63814	A7035	Every 6 months

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirFit™ P30i

The Resmed AirFit™ P30i features a top-of-head tube connection and SpringFit™ frame that adapts to the face, with dual-wall nasal pillows that flex with movement to help deliver a stable, comfortable seal in any sleep position.

Minimalist

Freedom

Versatile Fit

Nasal pillows mask



Components



How to fit the AirFit P30i | For complete user instructions, refer to the AirFit P30i user guide.



1 Pull the frame and headgear over the head until the elbow is positioned on top and the headgear is behind the head. Align the nasal pillows into the nostrils.



2 Tighten or loosen the headgear until the nasal pillows fit comfortably in the nostrils. If needed, feed longer hair through the back of the headgear.



3 Squeeze the side buttons on the elbow and detach from the frame. Connect the air tubing from the device to the elbow, then reattach to the frame.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, three pillows sizes and headgear) - Starter pack, small frame—small/medium/large pillows - Starter pack, standard frame—small/medium/large pillows	63851 63850	A7034 + A7035	—
Frame system (includes frame and pillows) - Small frame—small/medium/large pillows - Standard frame—small/medium/large pillows	63857/63855/63853 63856/63854/63852	A7034	Every 3 months
Pillows - Small - Medium - Large	63861 63862 63863	A7033	Every 2 weeks
Headgear Standard	63814	A7035	Every 6 months

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

AirFit™ F20

AirFit F20 for Her | AirFit F20 Non Magnetic

The Resmed AirFit™ F20 and AirFit F20 for Her feature an InfinitySeal™ silicone full face cushion designed to adapt to a wide range of facial profiles and deliver a reliable seal across therapy pressures.^{1,2,3}

Minimalist


Freedom

Versatile Fit

Full face mask



Components

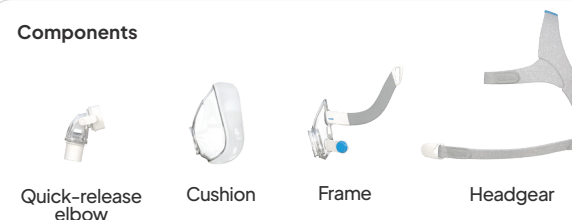

How to fit the AirFit F20 | For complete user instructions, refer to the AirFit F20 user guide.



1 Twist and pull both mask clips away from the frame. With the Resmed logo on the headgear facing outwards and upright, hold the mask against the face and pull the headgear over the head.



2 Bring the headgear straps into position and attach the mask clips to the frame. If needed, feed longer hair through the back of the headgear.



3 Undo fastening tabs on both headgear straps and pull the straps evenly until the mask feels stable and comfortable. Reattach the fastening tabs.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, cushion and headgear) - AirFit F20—small/medium/large - AirFit F20 for Her—small/medium - AirFit F20 Non Magnetic—small/medium/large	63400/63401/63402 63403/63404 64028/64029/64030	A7030 + A7035	—
Frame system (includes frame and cushion) - AirFit F20—small/medium/large - AirFit F20 for Her—small/medium - AirFit F20 Non Magnetic—small/medium/large	63460/63461/63462 63463/63464 64045/64046/64047	A7030	Every 3 months
Cushion - Small - Medium - Large	63467 63468 63469	A7031	Every month
Headgear (includes magnetic clips*) - AirFit F20—small/standard/large - AirFit F20 for Her—small - AirFit F20 Non Magnetic—small/medium/large	63470/63471/63472 63473 64040/64041/64042	A7035	Every 6 months

*AirFit F20 and AirFit F20 for Her contain magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings. AirFit F20 Non Magnetic is a non-magnetic variant available to order.

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

¹ Resmed AirFit F20 internal Australian fitting study of 27 existing CPAP patients conducted between March–April 2016.

² Resmed AirFit F20 internal USA fitting study of 34 existing CPAP patients conducted April 2016.

³ Resmed AirFit F20 internal EU and APAC fitting study of 90 existing CPAP patients conducted June 2016.

AirTouch™ F20

AirTouch F20 for Her

The Resmed AirTouch™ F20 and AirTouch F20 for Her feature an UltraSoft™ memory foam full face cushion designed to contour to facial features for a uniquely soft, comfortable fit while helping reduce facial irritation and maintain seal performance.¹

Minimalist

Freedom

Versatile Fit

Full face mask



Components



How to fit the AirTouch F20 | For complete user instructions, refer to the AirTouch F20 user guide.



Twist and pull both magnetic clips away from the frame. With the Resmed logo on the headgear facing outwards and upright, hold the mask against the face and pull the headgear over the head.



Bring the headgear straps into position and attach the magnetic clips to the frame. If needed, feed longer hair through the back of the headgear.



Undo fastening tabs on both headgear straps and pull the straps evenly until the mask feels stable and comfortable. Reattach the fastening tabs.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, cushion and headgear) • AirTouch F20—small/medium/large • AirTouch F20 for Her—small/medium	63000/63001/63002 63003/63004	A7030 + A7035	—
Frame system (includes frame and cushion) • AirTouch F20—small/medium/large • AirTouch F20 for Her—small/medium	63021/63022/63023 63024/63025	A7030	Every 3 months
Cushion • Small • Medium • Large	63028 63029 63030	A7031	Every month
Headgear (includes magnetic clips*) • AirTouch F20—small/standard/large • AirTouch F20 for Her—small	63470/63471/63472 63473	A7035	Every 6 months

*The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

¹Resmed AirTouch F20 internal USA fitting study of new and existing CPAP patients, conducted April 2016 and August 2016

AirFit™ N20

AirFit N20 for Her

The Resmed AirFit™ N20 and AirFit N20 for Her feature an InfinitySeal™ silicone nasal cushion designed to adapt to a wide range of facial profiles and deliver a reliable seal across therapy pressures¹

Minimalist

Freedom

Versatile Fit

Nasal mask

Components



How to fit the AirFit N20 | For complete user instructions, refer to the AirFit N20 user guide.



Twist and pull both magnetic clips away from the frame. With the Resmed logo on the headgear facing outwards and upright, hold the mask against the face and pull the headgear over the head.



Bring the headgear straps into position and attach the magnetic clips to the frame. If needed, feed longer hair through the back of the headgear.



Undo fastening tabs on both headgear straps and pull the straps evenly until the mask feels stable and sits comfortably just over the nose. Reattach the fastening tabs.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, cushion and headgear) • AirFit N20—small/medium/large • AirFit N20 for Her—small	63503/63501/63502 63500	A7034 + A7035	—
Frame system (includes frame and cushion) • AirFit N20—small/medium/large • AirFit N20 for Her—small	63553/63555/63556 63554	A7034	Every 3 months
Cushion • Small • Medium • Large	63550 63551 63552	A7032	Every 2 weeks
Headgear (includes magnetic clips*) • AirFit N20—small/standard/large • AirFit N20 for Her—small	63560/63561/63562 63558	A7035	Every 6 months

*The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

¹Resmed AirFit N20 international fitting study of 159 existing CPAP patients, conducted Nov 2015.

AirTouch™ N20

AirTouch N20 for Her

The Resmed AirTouch™ N20 and AirTouch N20 for Her feature an UltraSoft™ memory foam nasal cushion designed to contour to facial features for a uniquely soft, comfortable fit while helping reduce facial irritation and maintain seal performance.¹

Components


Quick-release elbow with short tube


Cushion


Frame


Headgear



How to fit the AirTouch N20 | For complete user instructions, refer to the AirTouch N20 user guide.

1



2



3



Twist and pull both magnetic clips away from the frame. With the Resmed logo on the headgear facing outwards and upright, hold the mask against the face and pull the headgear over the head.

Bring the headgear straps into position and attach the magnetic clips to the frame. If needed, feed longer hair through the back of the headgear.

Undo fastening tabs on both headgear straps and pull the straps evenly until the mask feels stable and sits comfortably just over the nose. Reattach the fastening tabs.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, cushion and headgear) - AirTouch N20—small/medium/large - AirTouch N20 for Her—small	63903/63901/63902 63900	A7034 + A7035	—
Frame system (includes frame and cushion) - AirTouch N20—small/medium/large - AirTouch N20 for Her—small	63953/63955/63956 63954	A7034	Every 3 months
Cushion - Small - Medium - Large	63950 63951 63952	A7032	Every 2 weeks
Headgear (includes magnetic clips*) - AirTouch N20—small/standard/large - AirTouch N20 for Her—small	63560/63561/63562 63558	A7035	Every 6 months

*The mask contains magnets that may interfere with certain implants or medical devices. Please refer to the User Guide for complete information, including magnet contraindications and warnings.

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.

¹Resmed guided external clinical study of 25 evaluable, existing CPAP mask users from Mar. 15 to Apr. 5, 2018, in Bella Vista, AU who trialed Resmed AirTouch N20 and Resmed AirFit™ N20 at home for 7 nights each in place of their prescribed nasal mask.

Pixi™

Pediatric nasal mask

The Resmed Pixi™ adjusts at the top, side and bottom to allow flexibility to fit children of various ages, and is designed to sit away from the eyes and ears to reduce irritation and obstruction.

Systems and Components


Plug


Tube assembly


Frame


Cushion


Headgear



How to fit the Pixi | For complete user instructions, refer to the Pixi user guide.

1



2



3



Hold the mask gently against the face, ensuring the nose is fully enclosed by the cushion and the mouth is clear. Pull the headgear over the head until it sits across the middle of the back of the head, keeping the frame positioned at the front.

Bring the lower headgear strap behind the neck and attach it by hooking the quick-release clip onto the mask. If needed, feed longer hair through the back of the headgear.

Adjust the headgear straps by releasing the Velcro and gently tightening until comfortable. For best results, adjust the lower straps first and work upward.

Ordering information

Product	Part number	HCPC	Recommended replacement schedule**
Complete mask system (includes frame, cushion and headgear)	61030	A7034 + A7035	—
Frame system (includes frame and cushion)	61035	A7034	Every 3 months
Cushion—standard	61033	A7032	Every 2 weeks
Headgear—standard	61034	A7035	Every 6 months

**The recommended replacement schedule is a reflection of the Medicare accessories replacement schedule. The Medicare accessories replacement schedule does not state how often Medicare recipients should replace accessories, but instead outlines the usual maximum amount of accessories expected to be reasonable and necessary.



Cleaning Resmed masks at home

Daily/after each use

1. Disassemble the mask per instructions.
 - For AirTouch™ masks: If required, wipe the cushion with a CPAP wipe or equivalent and then allow to dry. Important – do not get the cushion wet.
2. Rinse frame and elbow under running water. Clean with a soft brush until dirt is removed, paying close attention to the vents.
3. Soak components in warm water with mild soap for up to 10 minutes, then shake in water.
 - Important – do not get AirTouch™ cushions wet. Clean by wiping with a CPAP wipe or equivalent, then allow to dry.
4. Brush the following areas:
 - Moving parts of the elbow and around the vent holes
 - Parts of the frame where arms connect
 - Inside and outside the frame where the elbow and cushion connect
5. Rinse components under water (excluding AirTouch cushions).
6. Remove excess water by:
 - Squeezing frame sleeves with clean towel
 - Squeezing frame arms
 - Shaking vent (AirFit™ P10)
7. Air dry out of direct sunlight.

Weekly

1. Disassemble mask per instructions, keeping magnets in place (if any).
2. Handwash headgear in warm water with mild liquid detergent.
3. Rinse headgear under running water, ensuring it is clean and detergent-free. Wash and rinse again, if needed.
4. Squeeze headgear to remove excess water.
5. Air dry out of direct sunlight.

For detailed cleaning instructions on specific mask models, refer to the individual mask user guide.

Warning

- Do not use aromatic-based solutions or scented oils such as eucalyptus or essential oils. Do not use bleach, alcohol or products with strong smells such as citrus. Residual vapors from these solutions can be inhaled if not rinsed thoroughly and may crack or damage the mask.
- Do not iron headgear as material is heat-sensitive and will be damaged.

Disinfection and sterilization

Resmed offers recommended and validated procedures for the cleaning, disinfection and sterilization of masks in accordance with ISO 17664. For the multipatient use of a Resmed full face mask, nasal mask or nasal pillows system ('mask') in a sleep lab, clinic or hospital, please refer to the product guide available at [Resmed.com](https://resmed.com). For the use of a Resmed mask by a single user in the home, refer to the User Guide, or visit Resmed.com/MaskSupport for current masks, or Resmed.com/MaskGuide for all other masks, including AirFit P10. For more details regarding disinfection and sterilization for AirFit P10 and legacy masks, please visit Resmed.com/CleaningGuide.

Limited warranty information

Product type	Description	Warranty
Devices	AirSense™ 11, AirCurve™ 11, AirSense 10, AirCurve 10 (excluding external power supply units)	2 years
Device accessories	Humidifiers	1 year
	Humidifier water tubs	90 days
	Humidifier cleanable water tubs	1 year
	ClimateLine™ tubes	90 days
	ClimateLineAir™ tubes	90 days
	SlimLine™ tubes	90 days
	Standard tubes	90 days
	Oximeters and device oximeter adapters	1 year
	Batteries for use in Resmed internal and external battery systems	6 months
	Battery accessories	2 years
	Data modules	1 year
Mask systems	Including frame, cushion, headgear and tubing (excluding single-use devices)	90 days
Mask cushion	AirTouch™ UltraSoft™ cushion	7 days
Other	Accessories (excluding single-use devices)	90 days
	Flex-type finger pulse sensors	90 days
	Clip-type finger pulse sensors	1 year
	Titration control devices	1 year
	Portable diagnostic/screening devices	2 years

Resmed Online Store (ROS)

Get 24/7 access to:

- Full product details
- Real-time pricing
- Inventory status
- Part numbers
- Reimbursement codes

At ROS, you can:


- Place orders
- Track shipments
- Access digital packing slips
- Initiate return merchandise authorizations (RMAs)
- Check warranty status
- View and pay invoices

New to ROS?

Sign up for free!



OnlineStore.Resmed.com

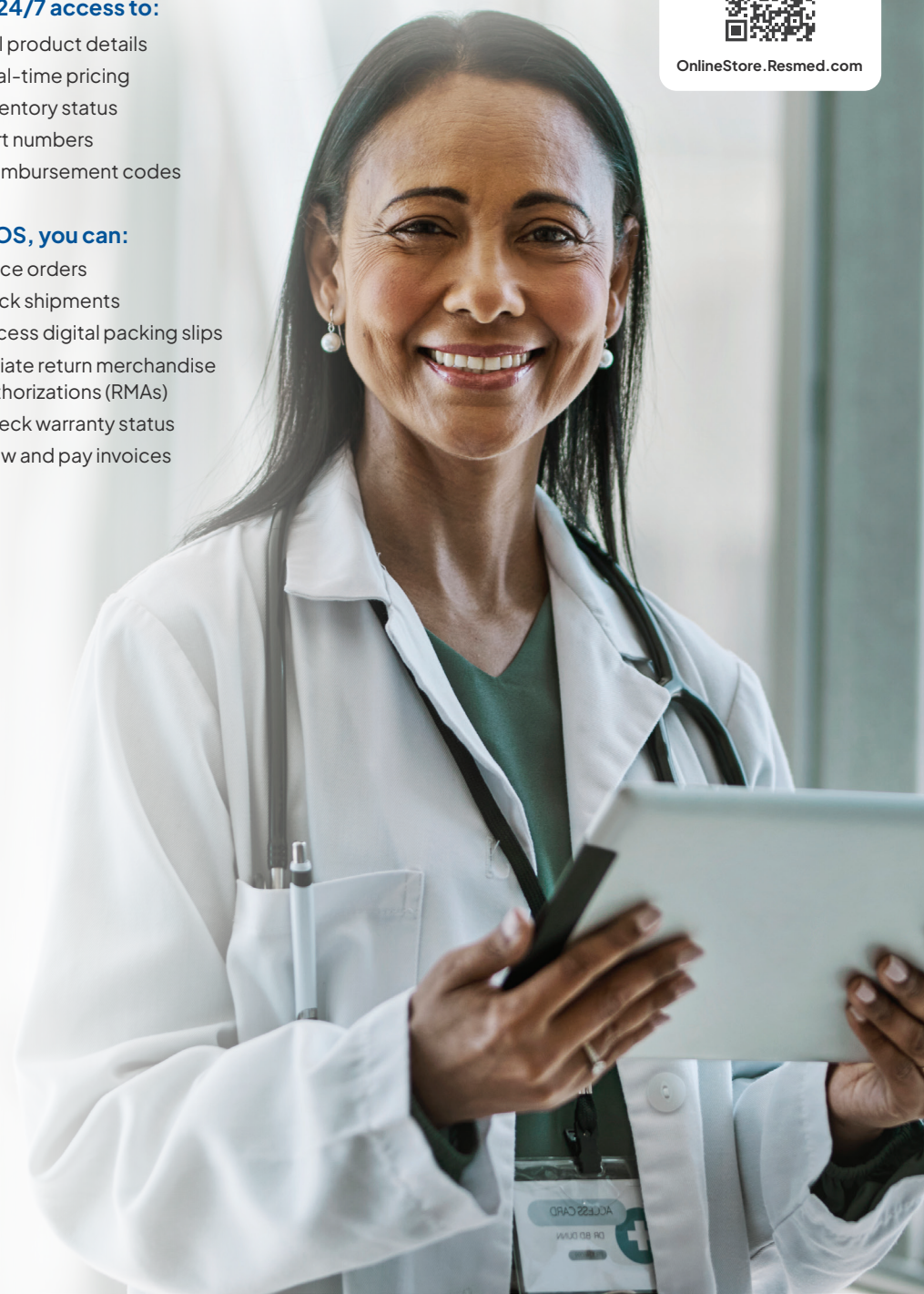



Resmed myAir™

The myAir™ app empowers patients to engage with their therapy, helping them start, acclimate, and adhere†† more successfully with an interactive guided setup, sleep therapy tracking, education, coaching tips** and more. It's all in myAir†.



Patients can download the myAir app through the App Store® and Google Play™ or online at myAir.Resmed.com.



† The myAir by Resmed app is available in English and Spanish in the US and in English and French in Canada. Some features of myAir are only available in the myAir app and with Air11 devices.

†† US Medicare adherence criteria: PAP use for ≥4 h per night on at least 70% of nights during a consecutive 30-day period during the first 90 days of initial usage.

** Coaching entails personalized feedback to encourage continuation of therapy. This is not a healthcare service and is not performed by licensed healthcare providers. Patients are encouraged to speak with their treating clinicians for any clinical symptoms or medical conditions.



Distributed by Resmed Corp, 9001 Spectrum Center Boulevard, San Diego, CA 92123 USA (858) 836-5000 or (800) 424-0737 (toll-free). See [Resmed.com](https://resmed.com) for other Resmed locations worldwide. AirCurve™ 11 bilevel PAP devices, AirSense™ 11 PAP devices, Air11™ accessories, AirCurve™ 10 bilevel PAP devices, AirSense™ 10 PAP devices, Air10™ accessories, S9™ accessories, AirMini™, Astral™ series ventilators, AirFit™ F40, AirFit™ N30, AirFit™ P10, AirTouch™ F30i Comfort, AirFit™ F30i, AirFit™ X30i, AirTouch™ N30i, AirFit™ N30i, AirFit™ P30i, AirFit™ F20, AirTouch™ F20, AirFit™ N20, AirTouch™ N20, and Pixi™ are trademarks and/or registered trademarks of the Resmed family of companies. Specifications may change without notice. For patent and other intellectual property information, see [Resmed.com/ip](https://resmed.com/ip).

© 2026 Resmed. 1013499/30 2026-04